
89-92-119-122-139-142 cm

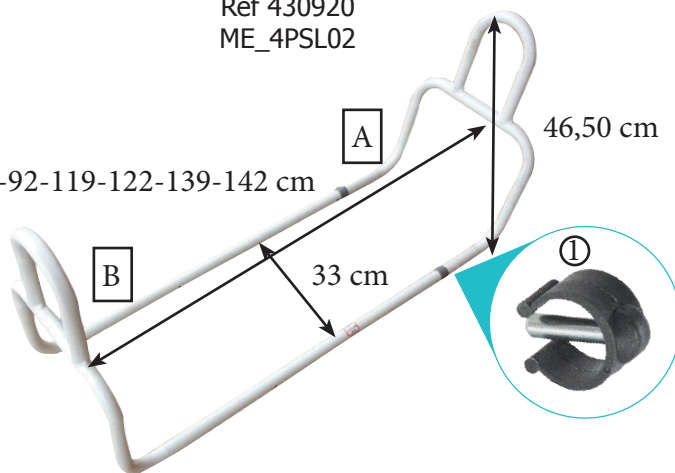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EN- Please read these safety instructions carefully before use, and keep them for future question.

1. Safety indications

- ♥ ALWAYS consult with your physician or supplier to determine proper adjustment and use of the device.
- ♥ Don't store outside and don't let under the sun (Beware : don't let the product in a car in case of high outside temperature !!)
- ♥ Check carefully all parts of this item (screws, tubes, clips...) and if all screws are tightened.
- ♥ To ensure that the safety level is kept to the highest possible standard, determined by its construction, this product should be serviced regularly (once a year) by specialist retailers.

- ♥ Any interference with parts of the product that are not described within the manual may cause damage, or endanger the person using this machine.
- ♥ In choosing the location of the apparatus, ensure a sufficient safety distance from any obstacles. The apparatus must not be mounted in the immediate vicinity of main passageways (paths, doorways, corridors).
- ♥ The equipment must be assembled with due care by an adult person. If in doubt call upon the help of a second person, if possible technically talented
- ♥ Use only as an assistive bed rail and in proper way.
- ♥ No modification must be made on this device. Use only original parts to replace the broken or defect one and let this be done by the technical staff.
- ♥ DO NOT hang anything on the rail.
- ♥ After assembling the product, make sure that it is securely locked and fixed before using.
- ♥ In case you didn't use your product for a long time, let it checked by the technical staff, before using.
- ♥ Always check stability before using.

2. Installation

Only specialists can adjust the product and teach you how to use it. A wrong use or adjustment could lead to overstress of the device or of the patient. The device must be adjusted individually by the technical staff to suit your needs. Ask the technical staff how to use this product and make sure that you understood clearly their explanation.

In case of doubt, don't hesitate to ask again.

Insert the [A] part onto the [B] part, lock with the clips (1).

Before using, check that the metal pin of the clip has gone through both pipes and exits from the outer pipe.

Place the bed rail under the mattress.

Check stability before using.

3. Cleaning

The raised seats has to be cleaned regularly with a mild cleaning agent and a soft cloth. It may be washed with soap and water or disinfected with non-chlorinated products. The product must be dried after each cleaning.

4. Materials

The product is made out of chromed steel and various plastic parts.

5. Disposal

Dispose of this product in accordance with your regional waste disposal regulations.

6. Technical specifications

This product has a 2 years warranty, from the date of purchase, against any manufacturing defect.

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